




www.gmedi.com




Endotracheal Tube Holder

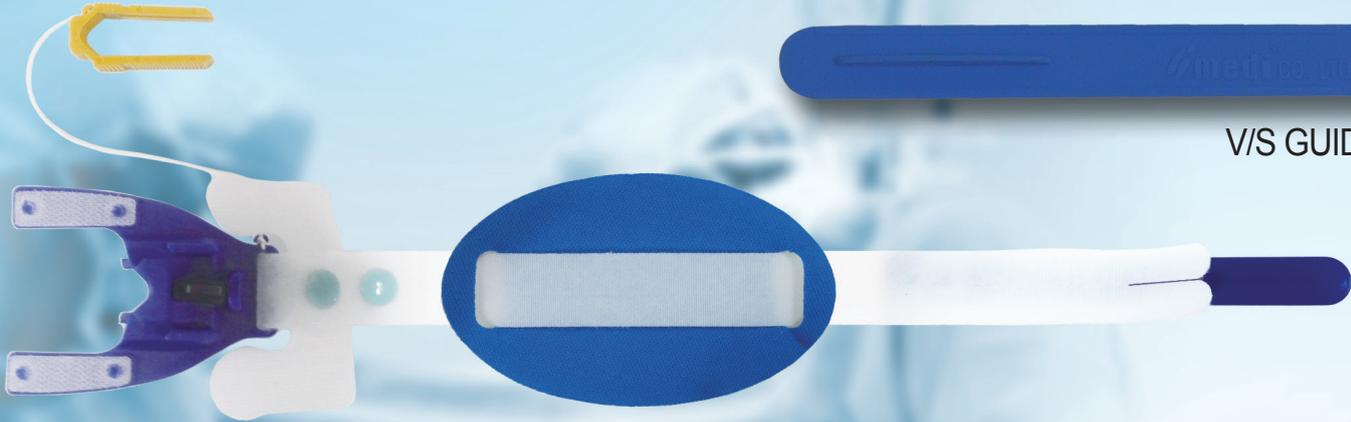

V/S GUIDER

ETG-02

ETG-03

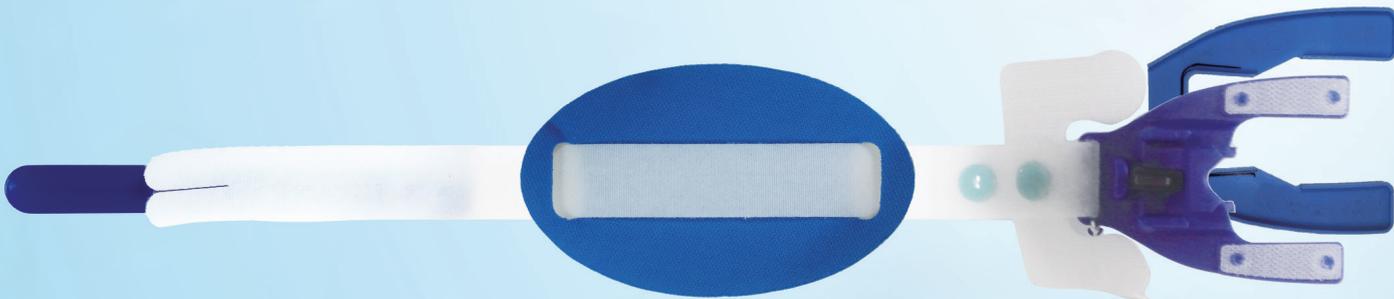

EXCHANGE CUSHION MODULE

V/S GUIDER

Very Accuracy to use and Safe!


<http://www.gmedi.com> <http://eblog.gobizkorea.com/gmedi> <http://kr109859445.trustpass.alibaba.com/>



gmedi® CO., LTD.

Endotracheal Tube Holder

ETG-02

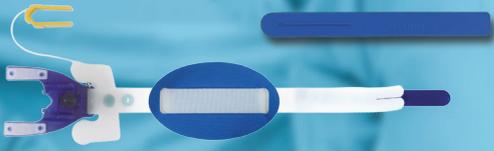

ETG-03



- Oral & Easy to Clean care
- Available two-way Left & Right Position
- Exchangeable Module (**ETG-03**)
 - Exchangeable Bobby Cushion & Saving Care Time

- 1 BLACK TWIST LOCK** – provides the extra security on tube position.
The lock pivots to hold the clip in place. (ETG-02, ETG-03)
- 2 LOCKING CLIP** – slides into place securing ET tubes(size4.5-14mm).
A patented gear rack quickly snaps into place to secure tubing. (ETG-02, ETG-03)
- 3 OFF SET POSITION** – provides wide access to the mouth.
Visualize, suction or perform oral care as needed without moving the ETG-02, ETG-03.
- 4 VELCRO STRAP** – quickly and easily.
Wraps around the patient's head to secure ETG-02 & ETG-03 to face.

● Application



After securing the "ETG", guide using the neck & back "Strap" moves.



Remove from the velcro strap guide




Velcro strap split in half by the "B" the fix



After cutting the velcro strap properly, the lock device "90 degree" to turn the lock

* The above specifications are subject to change without prior notice.

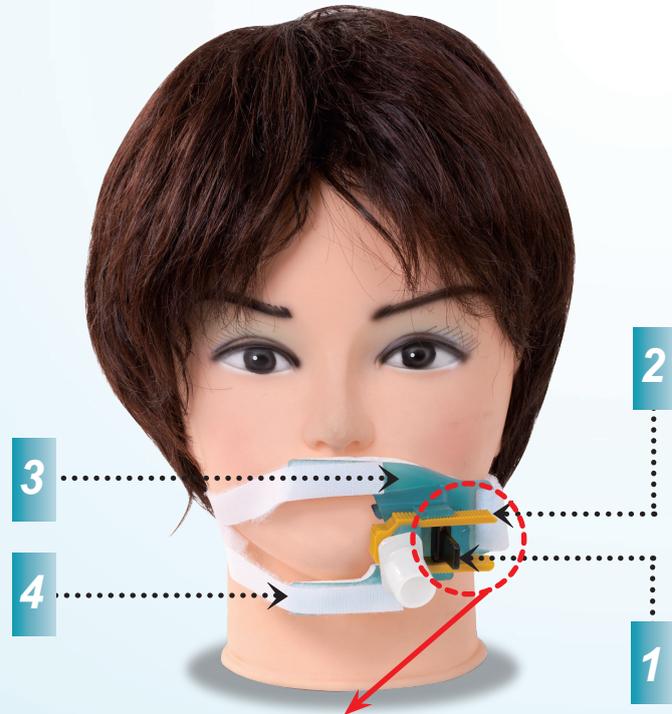

※ Caution ※
Lock Device is Keep the Direction of "90 degree"



Manufacture Address:

#401, Chunkang Apartment Shape Factory 402-3, Sindorim-dong, Guro-gu, Seoul, 152-887, Korea
TEL: +82-2-2632-9256 FAX: +82-2-2631-2659 E-mail: gmedir@yahoo.co.kr
<http://www.gmedi.com> <http://eblog.gobizkorea.com/gmedi> <http://kr109859445.trustpass.alibaba.com/>