



www.gmedi.com



Foley Cathater Holder

CTH-02


Foley Holder



Application method A



Application method B

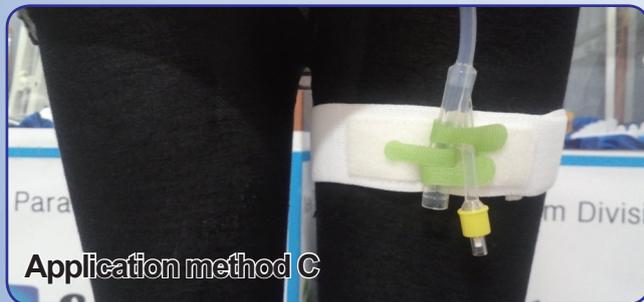

Application method C

Very Accuracy to use and Safe !

<http://www.gmedi.com> <http://eblog.gobizkorea.com/gmedi> <http://kr109859445.trustpass.alibaba.com/>



Gmedi® CO., LTD.

CTH-02 Foley Holder

CTH-02 Foley Holder

Gamma-ray Sterilized Product


Specify the Foley Catheter Holder and you can put an end to the problems associated with tape, and reduce the potential for infection and trauma.

Easy to put on and take off, the Foley Catheter Holder features a dual patented locking device that securely anchors the catheter in place. It helps minimize patient tampering, facilitates catheter traction, encourages mobility, and can be moved from leg to leg with total ease. The legband is comfortable, too, thanks to a unique stretch material that distributes compression evenly around the thigh to avoid a tourniquet effect.


A Velcro fastener engages the band at any point for a custom fit.

• Treatment Methods


1 Position the foley holder, one the leg

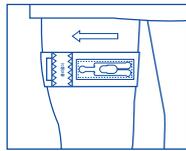


2 Place the foley catheter, one the g-locking middle side

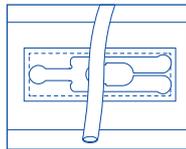


3 Move the g-locking left side and attachment

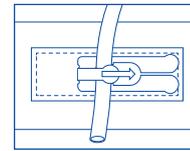


4 Move the g-locking right side and each ong attachment

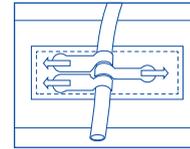


* The above specifications are subject to change without prior notice.

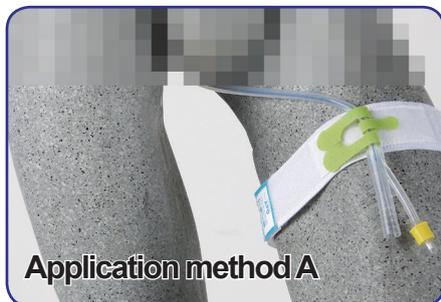

Application method A

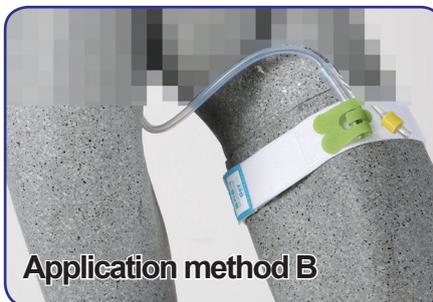

Application method B

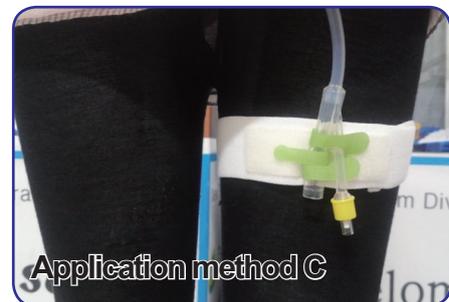

Application method C

High Soft Strap & Tighten



Manufacture Address:

#401, Chunkang Apartment Shape Factory 402-3, Sindorim-dong, Guro-gu, Seoul, 152-887, Korea
 TEL: +82-2-2632-9256 FAX: +82-2-2631-2659 E-mail: gmedir@yahoo.co.kr
<http://www.gmedi.com> <http://eblog.gobizkorea.com/gmedi> <http://kr109859445.trustpass.alibaba.com/>